


Easy to use

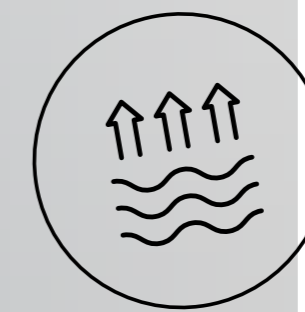

Note:

Please remove all sharp objects from your pockets before starting the massage, as these can damage the lying surface. For hygienic purposes, you should also remove your shoes before the massage. **Only lie down on the surface or get up when the massage jets are at a standstill.**

With Chipcard

If you use a chip card, the programme stored on it will run automatically. Please insert the card into the top of the control panel and press the Start button. Four therapeutical massage programmes are available on chip card:

- | | |
|------------------|----------------------------|
| 1. BODY BASIC | 2. SPECIAL BACK |
| 3. EXTENDED BODY | 4. INDIVIDUAL PROGRAMMABLE |

Without a Chip Card

Please enter the desired massage and press the Start button. You can alter the massage type, the massage area, the massage speed, the water pressure and the body size at any time.

- 1 Start / Stop**
Press »Start/ Stop« briefly in order to pause the programme. Press »Start/ Stop« again to resume the programme. Press and hold »Start/ Stop« for three seconds to end the programme.
- 2 Pulse massage**
The Pulse massage has a frequency of 100 bpm, which is optimal for the stimulation of the blood circulation.
- 3 Lumbar area massage**
The massage nozzles only move in the lumbar region.
- 4 Neck-/shoulder massage**
The massage nozzles only move in the neck and shoulder area.
- 5 Upper body massage**
For the upper body massage, the massage nozzles only move over the top half of the bed.
- 6 Whole body massage**
For the whole body massage, the massage nozzles move along the entire bed.
- 7 Lower body massage**
For the lower body massage, the massage nozzles only move over the lower half of the bed.
- 8 Point massage**
The massage nozzles move back and forth over a short distance. Select »Whole body massage«, wait until the nozzles have reached the desired body area and then press »Point massage«.
- 9 Section massage**
The massage nozzles move in an area of approx. 20 cm. Select »Whole body massage«, wait until the nozzles have reached the desired body area and then press »Zone massage«.
- 10 Water pressure**
If necessary, you can adjust the water pressure using the »Plus« and »Minus« buttons. First press the water pressure icon and then »Plus« or »Minus« accordingly.
- 11 Massage time**
Before starting the massage you can set the massage time by pressing this button and then »Plus« or »Minus« accordingly.
- 12 Chip card slot**
Please insert the programme or payment chip card into the top of the control panel and press the Start button (1).
- 13 Programming**
Press this button to store individually programmed massage programmes on chip card.

